

# ChM<sup>®</sup>

5.0 ChM Locked Plating  
5.0 ChLP system


5.0ChLP clavicular hook plate

3.4025; 3.4026







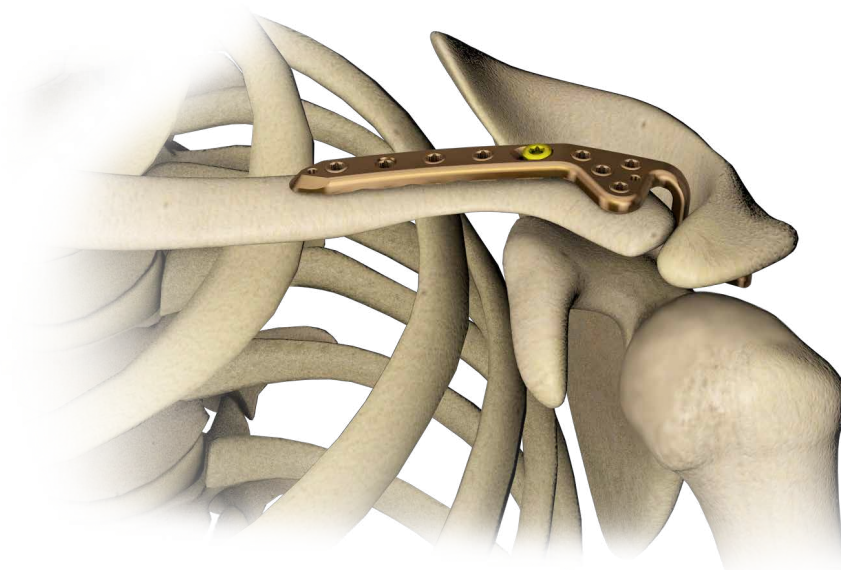


























3.4027; 3.4028

3.4029; 3.4030

- *SURGICAL TECHNIQUE*
- *IMPLANTS*
- *INSTRUMENT SET*



## SYMBOLS DESCRIPTION

	Titanium or titanium alloy		H length [mm]
	Cobalt		Angle
	Left		available lengths
	Right		Available number of holes
	Available versions: left/right		Thickness [mm]
	Length		Scale 1:1
	Torx drive		Number of threaded holes in the shaft part of the plate
	Torx drive cannulated		Number of locking holes in the plate
	Hexagonal drive		Variable angle
	Hexagonal drive cannulated		Cortical
	Cannulated		Cancellous
	Locking		Available in sterile/ non- sterile condition
	Diameter [mm]		Refer to surgical technique
	Caution - pay attention to a special procedure.		
	Perform the activity under X-Ray control.		
	Information about the next stages of a procedure.		
	Proceed to the next stage.		
	Return to the specified stage and repeat the activity.		
	Before using the product, carefully read the Instructions for Use. It contains, among others, indications, contraindications, side effects, recommendations and warnings related to the use of the product.		
	The above description is not a detailed instruction of conduct. The surgeon decides about choosing the operating procedure.		

**www.chm.eu**

Document No ST/80-502  
 Date of issue 04.07.2017  
 Review date P-001-07.12.2020

The manufacturer reserves the right to introduce design changes.  
 Updated INSTRUCTIONS FOR USE are available at the following website: ifu.chm.eu

<b>1. INTRODUCTION</b>	<b>5</b>
<b>2. IMPLANT DESCRIPTION</b>	<b>6</b>
<b>3. SURGICAL TECHNIQUE</b>	<b>8</b>
3.1. PATIENT'S POSITIONING	8
3.2. SURGICAL APPROACH	8
3.3. INITIAL FRACTURE REDUCTION	8
3.4. IMPLANT SELECTION	8
3.5. PLATE INSERTION	9
3.6. TEMPORARY PLATE STABILIZATION	9
3.7. CORTICAL SCREW INSERTION	10
3.8. LOCKING SCREWS INSERTION	10
3.10. WOUND CLOSURE	10
<b>4. SURGICAL PROCEDURES</b>	<b>11</b>
4.a. PROCEDURE OF TEMPORARY IMPLANT STABILIZATION	11
4.b. PROCEDURE OF CORTICAL SELF-TAPPING SCREW 3.5 [3.1306] INSERTION	12
4.c. PROCEDURE OF 5.0ChLP SELF-TAPPING SCREW 3.5 [3.5200] INSERTION	13
<b>5. POSTOPERATIVE PROCEDURE</b>	<b>14</b>
<b>6. IMPLANT REMOVAL</b>	<b>14</b>
<b>7. CATALOGUE PAGES</b>	<b>15</b>
7.a. INSTRUMENT SET	15
7.b. PLATES	17
7.c. SCREWS	18



## 1. INTRODUCTION

This surgical technique applies to 5.0ChLP locked plating system used for distal clavicular bone osteosynthesis. The plates are a part of the ChLP locked plating system developed by **ChM**. The presented range of implants is made of materials in accordance with ISO 5832 standards.

The system for the distal clavicular bone treatment includes:

- implants (*plates and screws*),
- instrument set used for conducting a surgical procedure,
- surgical technique.

### Indications

---

The plates are used to treat:

- Fractures of the acromial end of clavicle,
- Acute and inveterate acromioclavicular joint dislocations.

### Contraindications

---

#### Absolute:

- Health condition precluding surgery.
- Allergic reactions to the metal from which the implant is made.
- Active infection.

#### Relative:

- Stable lateral clavicular fractures.
- Weakened bone making it impossible to install/stabilize the implant properly.
- Abnormal perfusion of fracture area or surgical site.
- Excessive obesity.
- Lack of adequate tissue coverage.
- Psychiatric disorders or the disorders of the musculoskeletal system which may create a risk of fusion failure or complications in the postoperative period.
- Other medical conditions that exclude the potential benefits of the treatment.

### Plate selection and shaping

---

The plates are available in various hook height and length. This allows for optimal selection of the implant to the fracture type. Shaping of the plates in its epiphysial part is not allowed.



Before using the product, carefully read the Instructions for Use. It contains, among others, indications, contraindications, side effects, recommendations and warnings related to the use of the product.





The above description is not a detailed instruction of conduct. The surgeon decides about choosing the operating procedure.

---

## 2. IMPLANT DESCRIPTION

Clavicular hook plates are a part of 5.0ChLP system. This system includes also compatible locking screws. To facilitate the identification, both titanium plate and screws are brown anodized.

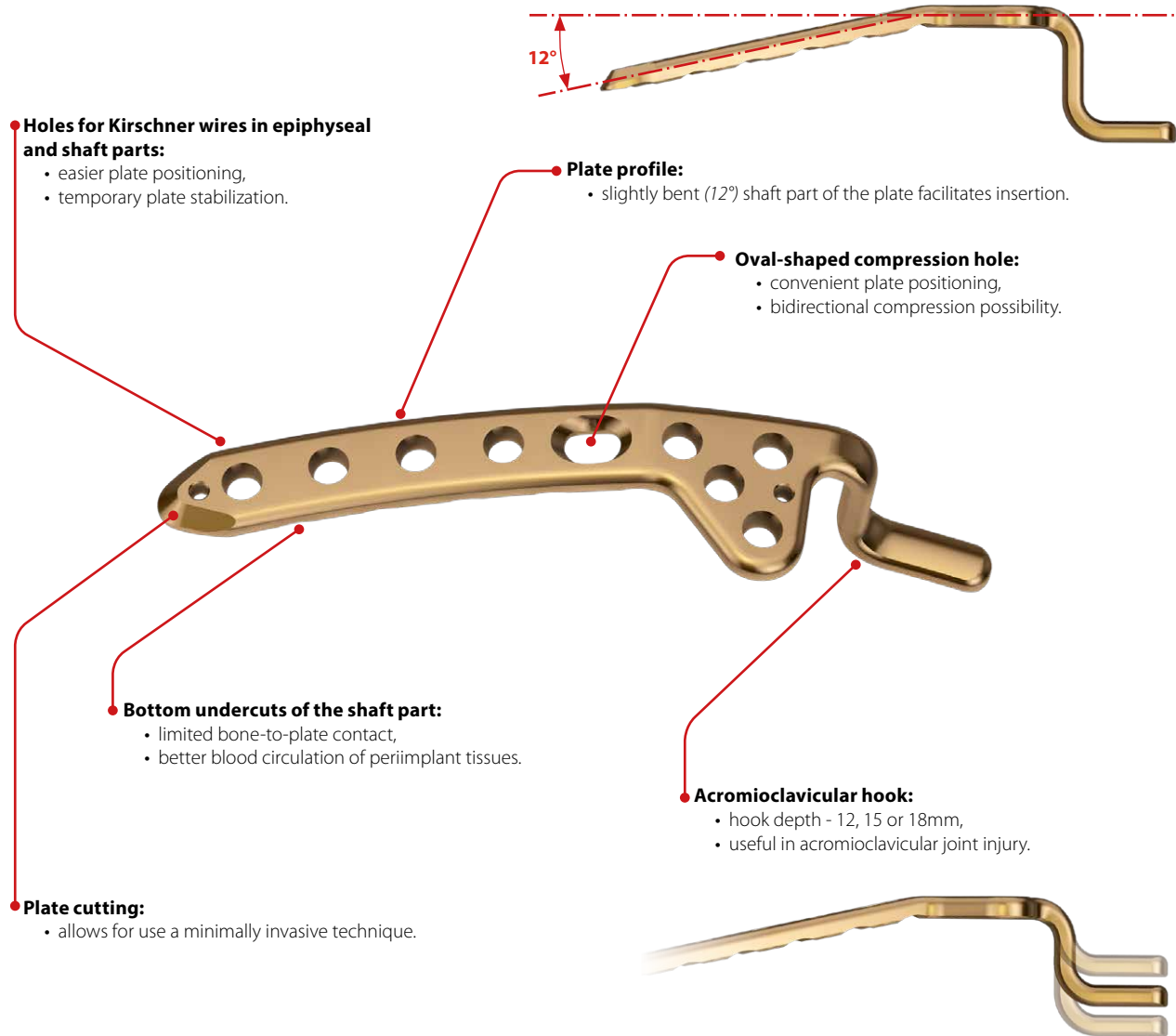

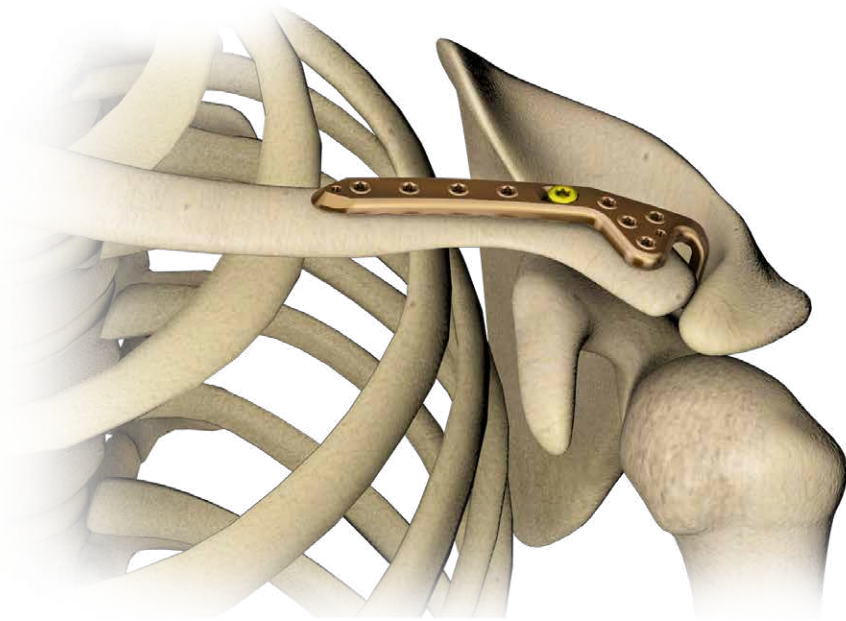


### 3. SURGICAL TECHNIQUE

This surgical technique defines steps when a fracture of the acromial end of clavicle or acromioclavicular joint dislocation appears.

#### 3.1. PATIENT'S POSITIONING

It is recommended to position a patient in a beach-chair position. Make sure the position allows taking adequate X-Ray image.

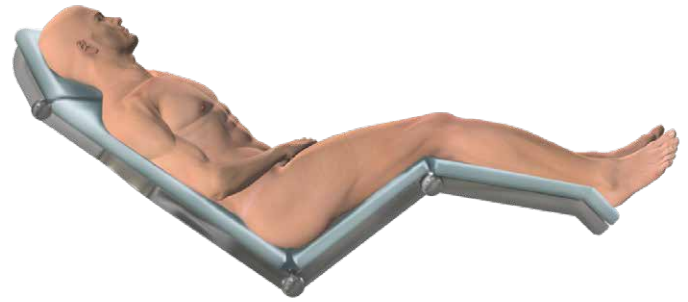

#### 3.2. SURGICAL APPROACH

**a. Fracture:** Expose the fracture site.

**b. Dislocation:** Expose and "clean" the joint.



**Dostęp przednio-górny typu „cięcie szablą” (ang. sabre cut).**

#### 3.3. INITIAL FRACTURE REDUCTION

**a. Fracture:** Reduce initially the fracture. If need be, temporarily stabilize the bone fragments with Kirschner wires and/or reduction pliers. Fracture reduction 'to the plate' possible.

**b. Dislocation:** Reduce initially the dislocation. If need be, temporarily stabilize the acromioclavicular joint with Kirschner wire. Capsular and ligament reconstruction recommended.

#### 3.4. IMPLANT SELECTION

Select the right height of the hook using plate trial.

H [mm]	Left	Right
12	43.4025.505	43.4026.505
15	43.4027.505	43.4028.505
18	43.4029.505	43.4030.505

Attach two guide sleeves 5.0/2.8 **[40.5673.728]** into the threaded holes of the plate trial to facilitate handling.



40.5673.728






Determine the right height of the hook. The end of the hook should rest on the bottom surface of the acromion.

Confirm the anatomical position of the clavicle and the acromion and that there is no collision of the hook with the head of the humerus.

Remove the plate trial from the operating site. Having determined the correct hook height, select the length of the plate from the available range of implants.

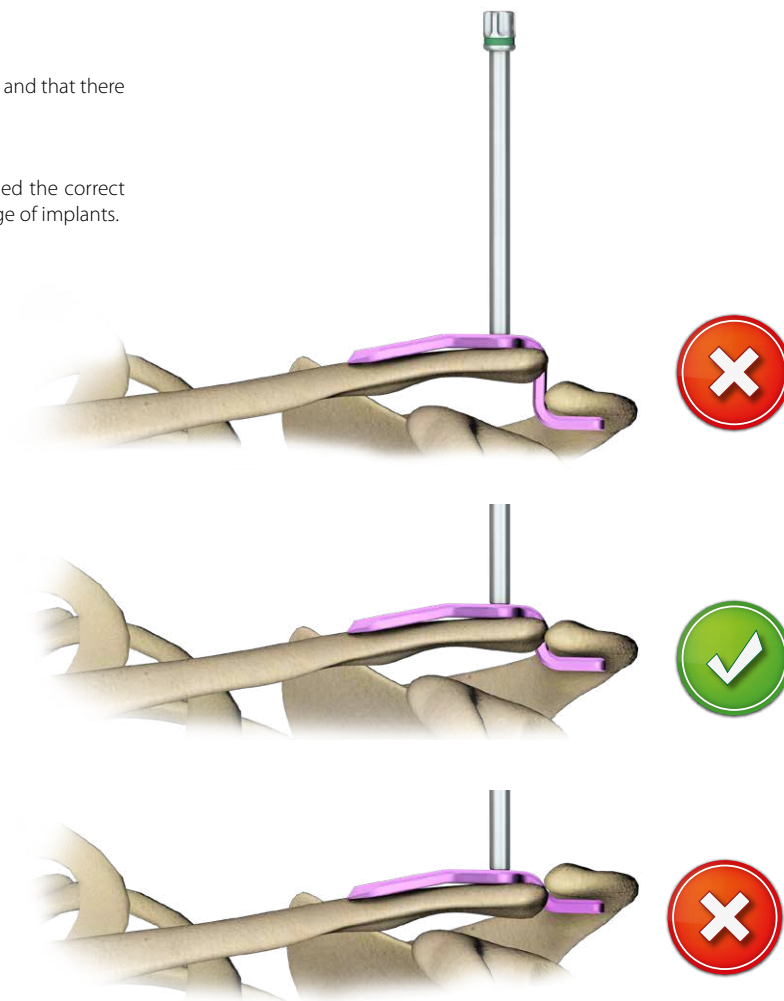


### 3.5. PLATE INSERTION

Position the implant correctly on the bone.

### 3.6. TEMPORARY PLATE STABILIZATION


The position of the implant may be stabilized by inserting Kirschner wires into appropriate holes or using setting-compressing screw (acc. to procedure 4a).

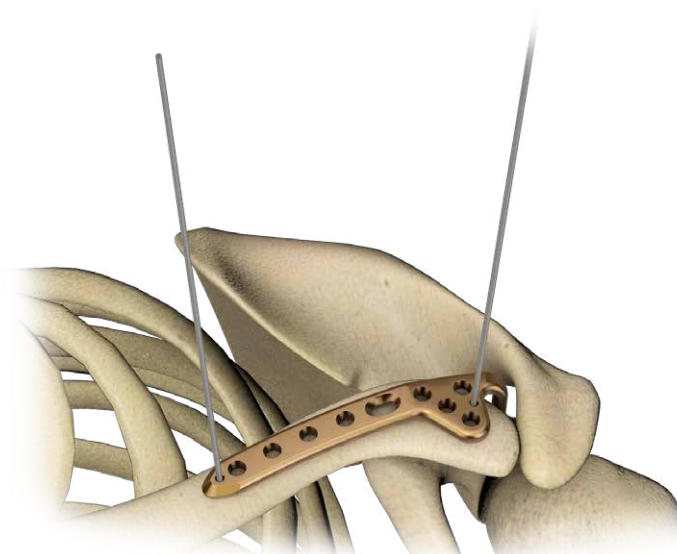

### 3.7. CORTICAL SCREW INSERTION


Insert cortical self-tapping screw 3.5 **[3.1306]** into the oval-shaped hole of the plate (*acc. to procedure 4b*)

The oval-shaped compression hole can be used to determine the correct position of the plate on the bone and its initial stabilization.

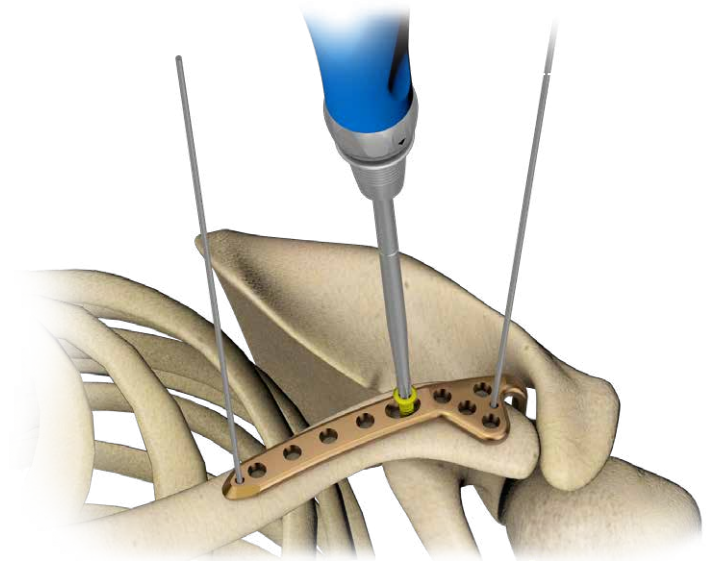

### 3.8. LOCKING SCREWS INSERTION

Insert 5.0ChLP self-tapping screw 3.5 **[3.5200]** of a suitable length, into the locking holes of the plate (*acc. to procedure 4c*).

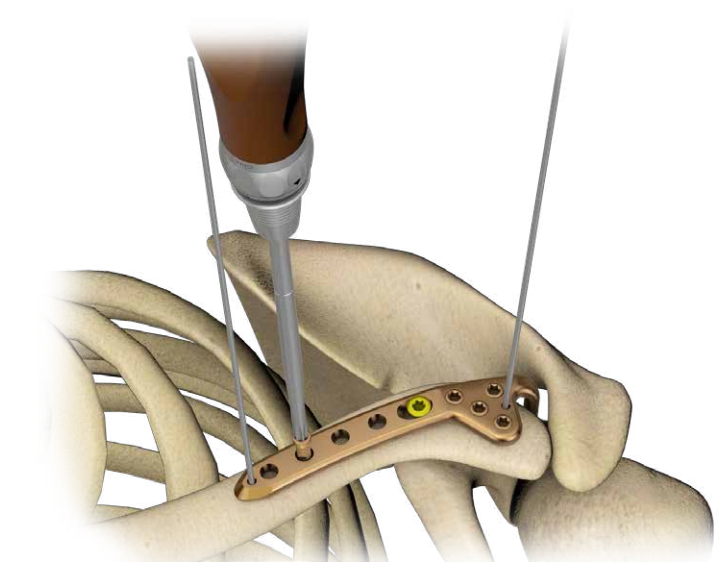

Insert the cortical screws 3.5 into the fracture before inserting the locking screws.



The doctor decides about the order and number of locking and cortical screws to be inserted.

### 3.10. WOUND CLOSURE

Before closing the wound, take an X-Ray image in at least two projections to confirm implant position and fracture reduction. Make sure all the screws are properly tightened and do not penetrate the joint surface.


Use appropriate surgical technique to close the wound.

## 4. SURGICAL PROCEDURES

### 4.a. PROCEDURE OF TEMPORARY IMPLANT STABILIZATION



#### Stabilization using Kirschner wires

- Stabilize temporary the implant inserting Kirschner wires 1.5/210 **[40.4592.210]** into dedicated holes in the plate.

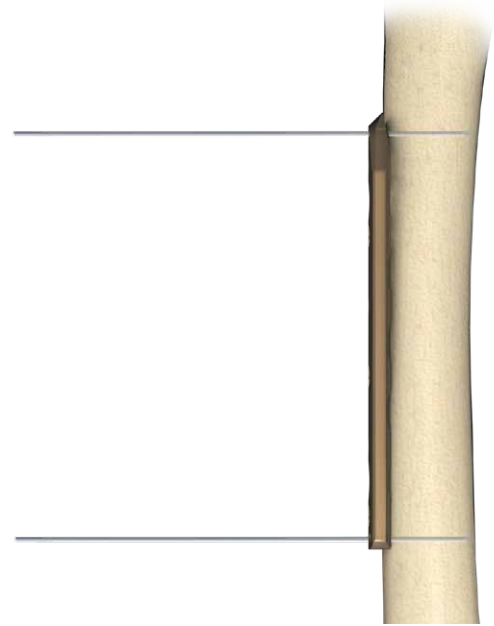



#### Stabilization in locking holes using Kirschner wires

- Insert guide sleeve 5.0/1.8 **[40.5673.718]** into the locking hole of the plate.
- Insert Kirschner wire **[40.4592.210]** through the guide sleeve 5.0/1.8 **[40.5673.718]**.

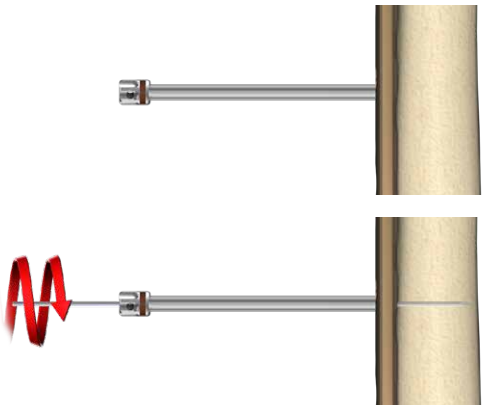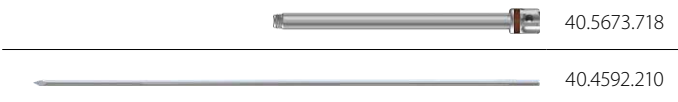

#### Stabilization using setting-compressing screw

- Insert guide sleeve 5.0/2.8 **[40.5673.728]** into the locking hole of the plate.
- Insert setting-compressing screw 2.8/180 **[40.5674.728]** through the guide sleeve 5.0/2.8 **[40.5673.728]**.
- Tighten the nut of the setting-compressing screw **[40.5674.728]** and push the plate to the bone.

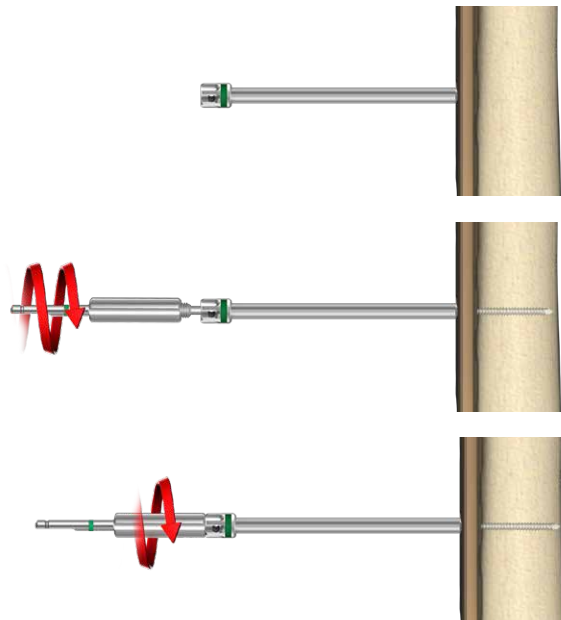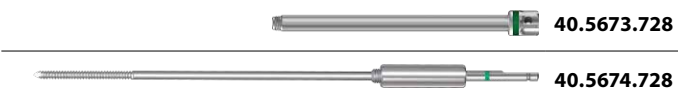

## 4.b. PROCEDURE OF CORTICAL SELF-TAPPING SCREW 3.5 [3.1306] INSERTION

### Compression guide positioning

Position the compression guide 2.5 [40.4804.725] in a desired position:



**NEUTRAL POSITION:** Push the guide to the plate. It will position itself so as neutral insertion of the screw is allowed.

**COMPRESSION POSITION:** Do not push the guide and move it to the edge of the compression hole. The hole drilled in this position allows compressive insertion of the screw.

**ANGULAR POSITION:** Angular position of the guide may also be applied.

NEUTRAL POSITION



COMPRESSION POSITION

ANGULAR POSITION

### Hole drilling

Perform a hole through both cortices for a cortical screw 3.5 insertion. For drilling, use drill with scale 2.5/210 [40.5912.212] and compression guide in a desired position.






### Measurement of hole depth

Insert depth measure [40.4639.550] into drilled hole until the hook of the measure rests against the outer surface of the second cortex.



### Screw insertion

Insert cortical screw using handle ratchet device [40.6654.000] and screwdriver tip T15 [40.5677.000].

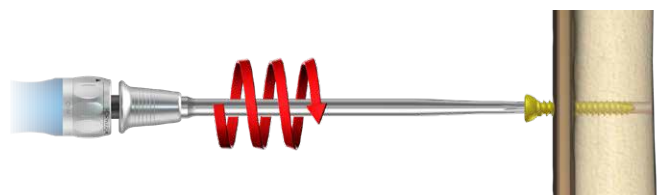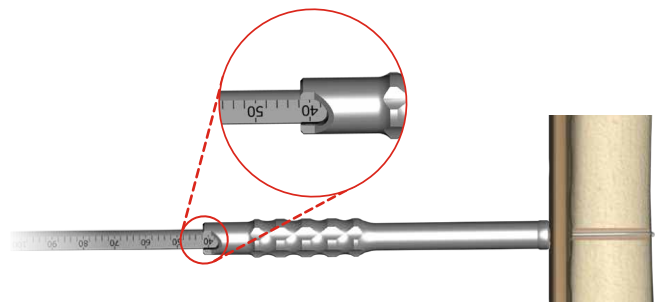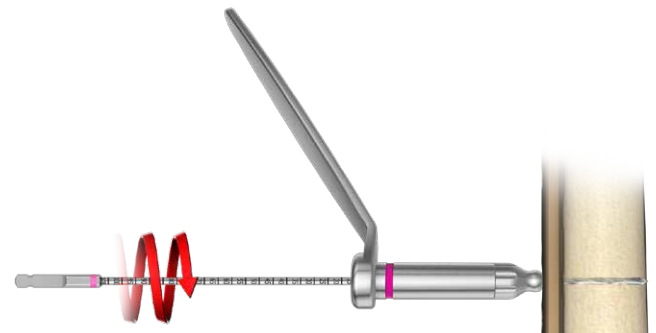

#### 4.c. PROCEDURE OF 5.0ChLP SELF-TAPPING SCREW 3.5 [3.5200] INSERTION

##### Guide sleeve insertion

- Insert guide sleeve 5.0/2.8 [40.5673.728] into a locking hole of the plate.



40.5673.728

##### Hole drilling

Drill using drill with scale 2.8/210 [40.5653.212] until desired depth is reached.



40.5653.212

##### Measurement of hole depth

OPTION I: Read the length of the screw from the drill measure [40.5653.212]



40.5653.212

OPTION II: or use screw length measure [40.5675.500].



40.5675.500

OPTION III: Having removed the guide sleeve 5.0/2.8 [40.5673.728], use depth measure [40.4639.550] to determine the length of a screw.



40.4639.550

##### Screw insertion

Remove the guide sleeve 5.0/2.8 [40.5673.728]. Use torque limiting ratchet handle 2Nm [40.6652.000] and screwdriver tip T15 [40.5677.000] to insert the locking screw.




40.6652.000



40.5677.000



The final tightening of the locking screw, especially when a drive is used, should always be performed with the use of torque limiting handle. Failure to use the torque limiting handle may lead to intraoperative and postoperative complications (*during later removal of the plate and locking screws*).

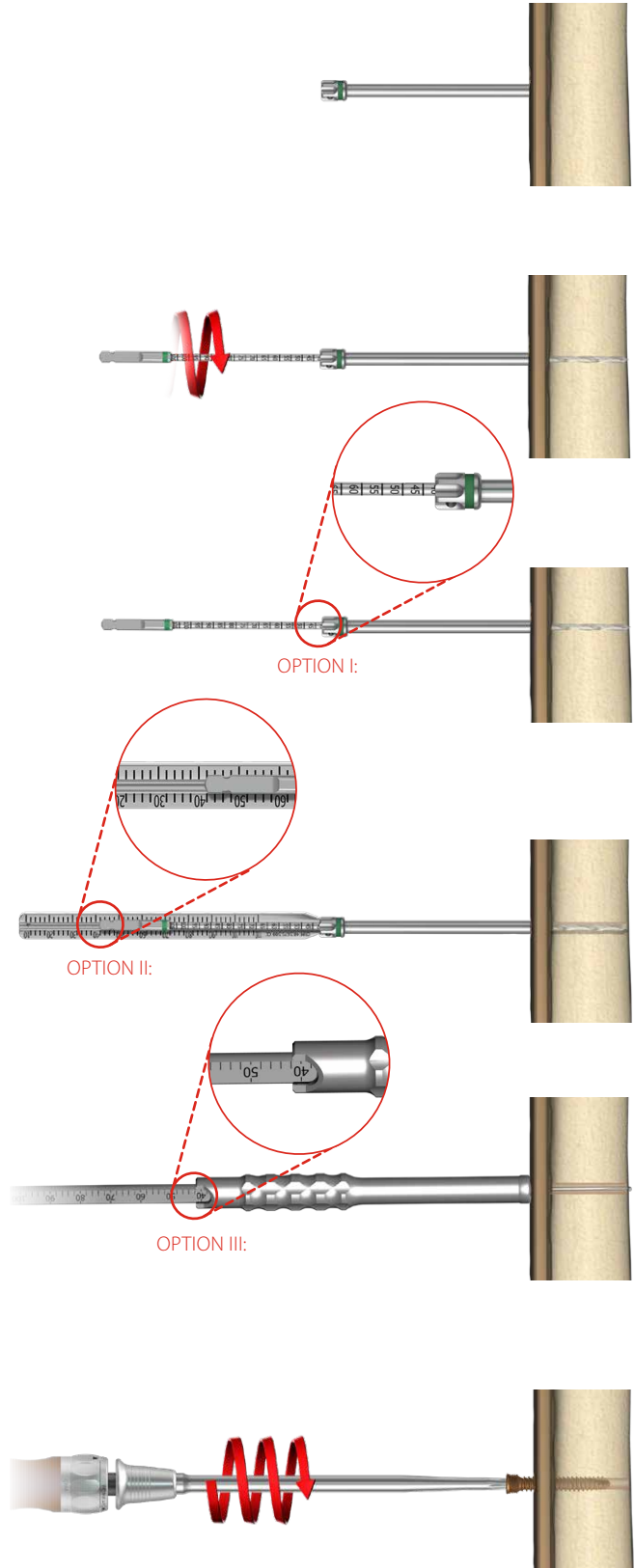

## 5. POSTOPERATIVE PROCEDURE

Introduce appropriate postoperative treatment that is determined by the physician. In order to avoid patient's movement limitations, introduce exercises as soon after surgery as possible. However, make sure that the limb is not fully loaded before fragments osteosynthesis is complete.

## 6. IMPLANT REMOVAL












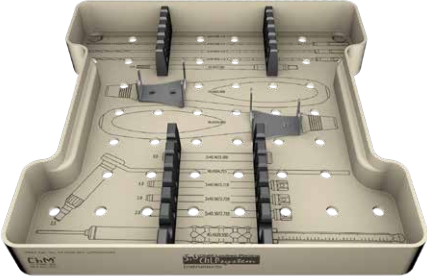

The physician decides about implant removal. In order to remove the implants from the body, unlock all the locking screws first and then remove them from the bone. This will prevent any rotation of the plate when removing the last locking screw.

## 7. CATALOGUE PAGES

## 7.a. INSTRUMENT SET

Instrument set for 5.0ChLP 4x4 1/2H

15.0205.201

	Name	Catalogue No.	Pcs
	Tray for 5.0ChLP instrument set 4x4 1/2H	14.0205.201	1
	Kirschner wire 1.5/210	40.4592.210	4
	Drill 1.8/210	40.2063.212	2
	Drill with scale 2.5/210	40.5912.212	2
	Drill with scale 2.8/210	40.5653.212	2
	Screwdriver tip T15	40.5677.000	1
	Torque limiting ratchet handle 2Nm	40.6652.000	1
	Handle ratchet device	40.6654.000	1
	Protective guide 7/5	40.5672.000	2
	Compression guide 2.5	40.4804.725	1
	Guide sleeve 5.0/1.8	40.5673.718	2
	Guide sleeve 5.0/2.8	40.5673.728	4
	Depth measure	40.4639.550	1

Instrument set for 5.0ChLP 4x4 1/2H

15.0205.202







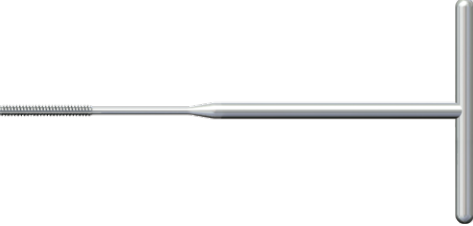




	Name	Catalogue No.	Pcs
	Tray for 5.0ChLP instrument set 4x4 1/2H	14.0205.202	1
	Setting-compressing screw 2.8/180	40.5674.728	1
	Screw length measure	40.5675.500	1
	Plates bender 5.0	40.4643.500	2
	Tripod screwdriver tip 5.0ChLP	40.6271.500	1
	T15 screwdriver tip with holder	40.6254.000	1
	Cortical tap HA 3.5 with handle	40.2548.200	1
	Tap 5.0ChLP-3.5	40.5661.000	1
<b>Optional instrument</b>			
	Torque connector 2Nm	40.5927.020	1

Plate trial

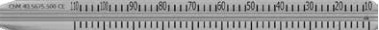


			
12	43.4025.505	43.4026.505	43.4028.505
15	43.4027.505	43.4028.505	43.4030.505
18	43.4029.505	43.4030.505	



4.5ChLP clavicular hook plate




12	5	3.4025.505	3.4026.505
	6	3.4025.506	3.4026.506
	7	3.4025.507	3.4026.507
15	8	3.4025.508	3.4026.508
	5	3.4027.505	3.4028.505
	6	3.4027.506	3.4028.506
18	7	3.4027.507	3.4028.507
	8	3.4027.508	3.4028.508
	5	3.4029.505	3.4030.505
18	6	3.4029.506	3.4030.506
	7	3.4029.507	3.4030.507
	8	3.4029.508	3.4030.508

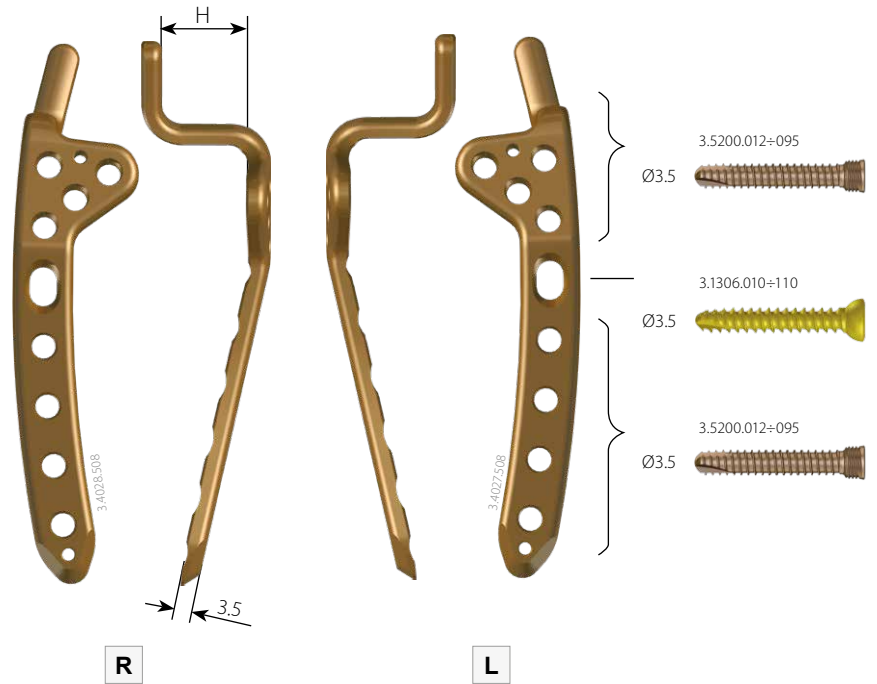

Tray for plates 5.0ChLP 3.4025-3.4030 4x4 1/2H

14.0205.403

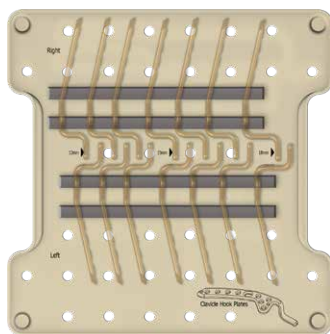

4.5ChLP Clavicular hook plate



Catalogue No.

3.4025.5xx ÷ 3.4029.5xx

Pcs

7

4.5ChLP Clavicular hook plate



Catalogue No.

3.4026.5xx ÷ 3.4030.5xx

Pcs

7

\* Tray does not include implants



## 5.0ChLP self-tapping screw 3.5



Len	Ti
12	3.5200.012
14	3.5200.014
16	3.5200.016
18	3.5200.018
20	3.5200.020
22	3.5200.022
24	3.5200.024
26	3.5200.026
28	3.5200.028
30	3.5200.030
32	3.5200.032
34	3.5200.034
36	3.5200.036
38	3.5200.038
40	3.5200.040
42	3.5200.042
44	3.5200.044
46	3.5200.046
48	3.5200.048
50	3.5200.050
52	3.5200.052
54	3.5200.054
56	3.5200.056
58	3.5200.058
60	3.5200.060
65	3.5200.065
70	3.5200.070
75	3.5200.075
80	3.5200.080
85	3.5200.085

## Cortical self-tapping screw 3.5



Len	Ti
10	3.1306.010
12	3.1306.012
14	3.1306.014
16	3.1306.016
18	3.1306.018
20	3.1306.020
22	3.1306.022
24	3.1306.024
26	3.1306.026
28	3.1306.028
30	3.1306.030
32	3.1306.032
34	3.1306.034
36	3.1306.036
38	3.1306.038
40	3.1306.040
45	3.1306.045
50	3.1306.050
55	3.1306.055
60	3.1306.060
65	3.1306.065
70	3.1306.070
75	3.1306.075
80	3.1306.080
85	3.1306.085



**ChM sp. z o.o.**

Lewickie 3b  
16-061 Juchnowiec Kościelny  
Poland

tel. +48 85 86 86 100

fax +48 85 86 86 101

chm@chm.eu

www.chm.eu



CE 0197